Artikelnummer: Z 01216

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Product number: Z 01216

Dear customer,
We are delighted that you have decided to purchase our **knee bandage**. If you have any questions, please contact our customer service via our website:

www.service-shopping.de

Proper use

- The product is suitable for application on the leg.
- Suitable to be worn when doing sports, on the go or at home.
- Only use the product for the intended purpose. Any other use is deemed improper.
- Defects caused by improper handling, damage or failure to observe the care notes are excluded from the warranty. This also applies to normal wear and tear.

**Safety notes**

- Risk of suffocation! Keep the product and the packaging material away from children and animals.
- The product is not suitable for therapeutic applications. Consult your family doctor or orthopaedist before application.
- If you feel persistent pain, immediately consult your family doctor.
- The product is not suitable for treating bone fractures or torn ligaments etc. Consult your physician in case of acute injuries.
- Store the product in a dry, clean area that is out of the reach of children and animals and that is not exposed to direct sunlight.

Material

80 % chloroprene rubber, 20 % polyester

Care

Hand wash

Do not bleach

Do not dry in the dryer

Do not iron

Do not dry clean

Use

- Pull the bandage over your foot and up to just below your knee. Tighten the Velcro straps. The acupressure point on the back should now rest on the back of your calf.
- The product can be worn both on the left and right leg and is suitable for all sizes. Observe the marks **L** for the left leg or **R** for the right leg in this respect. If you wear the product on your right leg, the **R** must be positioned on the top of the knee bandage and if you wear the product on your left leg, the **L** must be positioned on the top.
- The product can be worn both under and over your clothes.

Disposal

The packaging material is recyclable. Dispose of the packaging in an environmentally-friendly manner and make it available to the collection service for recyclable materials.

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Alle Rechte vorbehalten. | All rights reserved.
DS Produkte GmbH, Am Heisterbusch 1, 19258 Gallin, Deutschland (Allemagne | Duitsland) • ☎ +49 38851 314650 *)