04379

DE

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ab Seite 3

FR

Mode d'emploi

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EN

Operating Instructions

starting on page 25

NL

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vanaf pagina 69

Inhalt

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Contents

Information about your operating instructions	26
Intended use	27
Product description	28
SAD symptoms	28
Contraindications	29
Adverse effects	29
Safety instructions	30
Scope of supply	33
Device overview	33
Before initial use	34
Set-up and connection	34
Operation	35
Instructions for use	35
Cleaning and storage	36
Troubleshooting	37
Technical data	38
Safety and performance standards	39
Disposal	39
Normative references	40

Congratulations!

Dear Customer,

We are delighted that you have chosen this **LED light therapy lamp**.

If you have any questions about the device and about spare parts / accessories, please contact the customer service department via our website: www.dspro.de/kontakt

Information about your operating instructions

Before using this device for the first time, please read the operating instructions carefully and keep them for future reference. They are an integral part of the device. If you pass on the device to others, make sure to also provide the operating instructions.

The manufacturer and importer do not accept any liability if the information in these operating instructions is not complied with.

Explanation of the symbols

The following symbols are used in the operating instructions as well as on the packaging, the device and any accessories:

Danger symbols: These symbols indicate a risk of injury. Read through the associated safety instructions carefully and follow them.

Read the operating instructions before use

Manufacturer

Concordance with
Directive 93/42 EEC

Warning about non-ionising radiation
(electromagnetic fields)

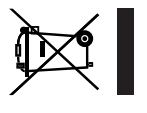
Date of manufacture

Specification of the batch number

Supplementary information

	EU Representative		Protection class II device
IP21	Ingress Protection Rating		Polarity of the coaxial connector (mains adapter)
	Only use indoors!		Efficiency class V
	Electrical and electronic devices must be properly disposed of in accordance with the WEEE Directive		

Explanation of signal words

- DANGER** indicates the presence of a hazard that will cause serious injuries and danger to life if ignored.
- WARNING** indicates the presence of a hazard that *could* cause serious injuries and danger to life if ignored.
- NOTICE** indicates the presence of a hazard that will or could cause damage to property if ignored.

Intended use

- The device is designed to prevent or reduce the effects of seasonal affective disorder (SAD), also known as winter depression.
- The device is intended to be used only on humans.
- The device is **not** suitable for medical or therapeutic purposes. It is not a replacement for medically required treatment.
- No special training is required to use the device.
- The device does not require any calibration, preventive inspection or maintenance measures.
- The device is not equipped with any parts that can be replaced or must be maintained by the user.
- Use the device only as described in the operating instructions. Any other use is deemed to be improper.

- The device is protected against unauthorised use. Never modify the device without the manufacturer's approval. If you modify the device, suitable inspection and test procedures must be carried out in order to ensure that the device is still safe for use.
- The device is intended for personal domestic use only and is not suitable for commercial applications.
- All defects caused by improper handling or damage are excluded from the warranty. This also applies to normal wear and tear.

Product description

The autumn and winter months are known for their short days and sparse hours of sunshine. This causes many people to suffer from a so-called seasonal affective disorder (SAD), more commonly known as winter depression. The lack of sunlight affects the release of chemical substances in the brain, which in turn has a negative impact on the functions of the hypothalamus. Furthermore, it is assumed that a lack of sunlight triggers the production of the hormone melatonin. Increased melatonin levels make us feel tired and sleepy.

The bright rays of the light therapy lamp simulate sunlight. The light therapy lamp also has the advantage that the emitted light does not contain any harmful UV rays. The additional exposure to light helps the body to regulate its melatonin production and balance its circadian rhythm. This not only combats the fatigue but also the winter depression as a whole.

SAD symptoms

- Listlessness
- Weariness
- Excessive tiredness
- Concentration problems
- Lack of energy
- Excessive need for sleep
- Decreased activity level

Contraindications

 **If you have any health concerns, suffer from health problems or belong to a risk group, consult your doctor before using the device.**

- The following groups of people must take special care when using the device:
 - People suffering from severe depression
 - People who have recently had eye surgery and who show a sensitive reaction to bright light
 - People suffering from certain eye diseases, e.g. cataracts or glaucoma
 - People suffering from an optic nerve disorder
 - People suffering from vitreous body inflammation
 - People with high blood pressure
 - People with photosensitive skin, e.g. systemic lupus erythematosus

The following groups of people should consult their doctor before using the device:

- People who take therapeutic drugs such as antidepressants, psychotropic drugs, malaria medication, painkillers or antihypertensive drugs may use the device only after consulting their doctor.
- Diabetics and people suffering from retinal diseases may only use the device after consulting their eye specialist
- If you experience pain or a feeling of discomfort during or after use of the device, stop exposure immediately and consult your doctor before using the device again.

Adverse effects

The following adverse effects are possible:

- Headache
- Eye strain
- Sleeping problems
- Nausea

The following measures may reduce the occurrence of adverse effects:

- Increase the distance between yourself and the device
- Avoid using the device late in the evenings or before going to bed
- Take a break from the sessions for a few days until the adverse effects have subsided before using the device again

Safety instructions

- This device should not be used by people (including children) with impaired sensory or mental faculties or who do not have adequate experience and / or knowledge of such devices, unless they are supervised by a person responsible for their safety, or have received instructions from such a person on how to use the device.
- **Children must not play with the device.**
- Cleaning and **user maintenance** may not be performed by **children**, unless they are supervised.
- Children younger than 8 years of age and animals must be kept away from the device and the mains cable.
- Do not use the mains adapter with another device and never use the device with another mains adapter.
- The device must always be disconnected from the mains power when left unattended and before cleaning it.
- In the interest of your own safety, check the device, the mains cable and the mains adaptor for damage every time before use. Only use the device if it is in good working order. If the mains adapter, mains cable or device is damaged, contact the customer service or a specialist dealer, as repairs can only be carried out with a special tool.

- Do not use a damaged device.
- Never try to repair the device yourself. This could cause serious injuries. In case of failure to follow the above, the guarantee will become void.

DANGER – Risk of electric shock

- Use and store the device only in enclosed areas. Do not operate it outdoors or in closed areas with high humidity.
- Never immerse the device in water or other liquids! Ensure that the device, the mains cable and the mains adapter cannot fall into water or get wet.
- Should the device fall into water, switch off the power supply immediately. Do not attempt to pull the device out of the water while it is still connected to the mains power!
- Never touch the device, mains cable or mains adapter with wet hands when these components are connected to the mains power.
- Do not use the device if it has malfunctioned or been dropped in water. Have it checked in a specialist workshop before using it again.

WARNING – Risk of injury

- **Risk of suffocation!** Keep children and animals away from the device and packaging material.
- **Risk of strangulation!** Make sure that the mains cable is always kept out of the reach of small children and animals.
- **Risk of dazzling!** Never look directly into the light source for a prolonged period! This could damage your eyes.
- **Risk of tripping!** Always lay the mains cable in such a way that it does not present a tripping hazard.

WARNING – Fire hazard

- Do not use the device in rooms containing easily ignitable dust or poisonous and explosive fumes.
- Do not operate the device in the vicinity of combustible material.
- Do not cover the device during operation in order to prevent it from catching fire.
- Unwind the mains cable fully before connecting the mains adapter.

NOTICE – Risk of damage to material and property

- When the mains adapter is removed from the plug socket, always pull on the mains adapter and never the mains cable.
- Do not pull or carry the device by the mains cable. Do not wrap the mains cable around the mains adapter or the device (this may cause the cable to break).
- Use only original spare parts from the manufacturer in order to guarantee that there is no interference that may prevent the device from working and to avoid damage.
- Never place heavy objects on the device or the mains cable.
- Magnetism can impair or damage the functions of electronic devices, data storage media and watches as well as credit cards and other similar items. Keep these items out of reach of the stand's storage compartment.
- Protect the device from heat, naked flames, direct sunlight, sub-zero temperatures, persistent moisture, wet conditions and impacts.

Scope of supply


PLEASE NOTE!

- Keep the original packaging during the warranty period of the device so that the device can be properly packaged up if it needs to be returned. The warranty claim will lapse if any damage is caused during transport.
- LED light therapy lamp
- Mains adapter
- Stand
- Storage bag
- Operating instructions

Check the items supplied for completeness and transport damage. If you discover any damage to the components, do not use them (!), but contact the customer service department.

Device overview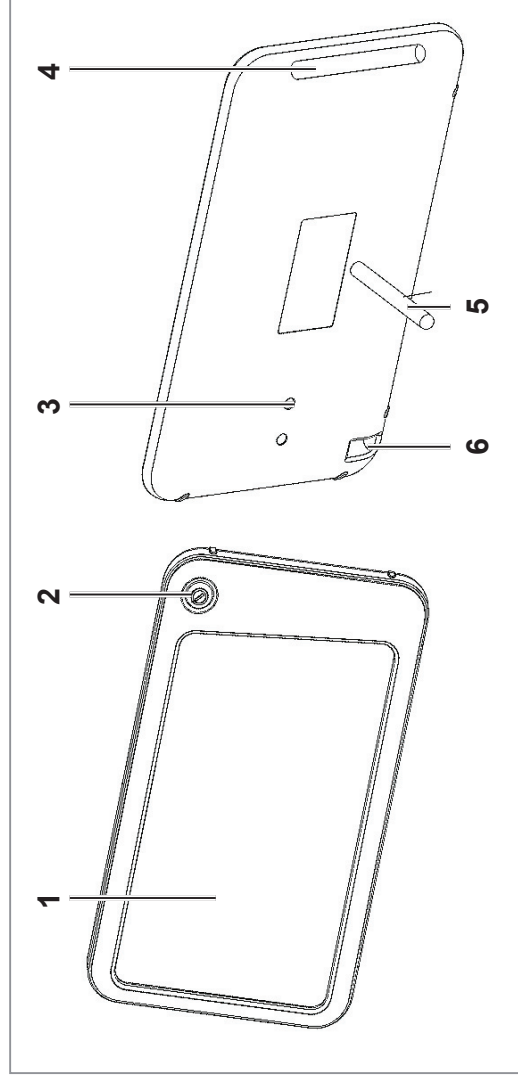

- 1 Illuminated area
- 2 On / Off switch
- 3 Recesses for stand
- 4 Storage compartment for stand (magnetic)
- 5 Stand
- 6 Connecting socket for the barrel connector of the mains adapter

Before initial use

PLEASE NOTE!

- The mains adapter may only be connected to a properly installed socket with safety contacts. The socket must also be readily accessible after connection so that the connection to the mains can quickly be isolated. The mains adapter may only be connected to the mains voltage indicated on the rating plate.
 - Only use suitable extension cables whose technical data matches that of the device and the mains adapter.
 - The mains cable may not hang down from the surface on which the device is placed in order to prevent the device from being pulled down.
 - Remove the mains adapter from the plug socket if a fault occurs during operation or if a thunderstorm is expected.
 - Make sure that the mains cable is not squashed, bent or laid over sharp edges and does not come into contact with hot surfaces.
-
- Remove any protective films and stickers that may be present from the device. **Please note! Never remove the rating plate or any warnings.**
 - Before using the device, it is important that you make yourself familiar with the operating instructions as well as with the indications, contraindications, safety warnings and safety measures. However, reading the operating instructions is not a substitute for consultation with a doctor!

Set-up and connection

1. Remove the stand from the storage compartment.
2. Insert the stand into the provided recesses.
3. Place the device on a level, stable and dry surface so that it faces the side of your face at a distance of approx. 20 – 30 cm.
4. Connect the barrel connector of the mains adapter to the connecting socket on the device and insert the mains adapter into an easily accessible plug socket.

Operation

The device has two different light intensity levels.

1. Press the On/Off switch to switch on the device. The device will light up at high intensity.
2. Press the On/Off switch again to set the device to the lower intensity level.
3. Press the On/Off switch again to switch off the device.

Instructions for use

- You may experience mild headaches or eye pain the first times you use the device – these should improve over time. If this is not the case, reduce the length of the session until it is pleasant for you.
- While sitting in front of the lamp, you can carry on with your normal activities like reading, telephone calls or working.
- The period of exposure depends on the set intensity level and the distance between the device and your face. The nearer the device is positioned to your face, the shorter the period of exposure. Please note the guideline figures for the period of application set out below:

Distance	Exposure period at high intensity level	Exposure period at low intensity level
30 cm	approx. 2 hours	–
20 cm	approx. 1 hour	approx. 2 hours

- For maximum effect, we recommend using the device over a period of at least seven consecutive days during the winter months. This period can be extended to meet your individual requirements.

Cleaning and storage

PLEASE NOTE!

- Make sure that no water enters the device or is spilled over the device.
 - Do not use any corrosive or abrasive cleaning agents to clean the device. These can damage the surface.
-
1. Allow the device to cool down entirely after use.
 2. Disconnect the device from the mains power before you clean it.
 3. Clean the device with a soft, slightly damp cloth and if required a little mild washing-up liquid.
 4. Thoroughly dry off the device before putting it away.
 5. Store the device in a dry place that is protected from sunlight and is not accessible to children and animals. Never leave the device connected to the mains.

Troubleshooting

Do not use the device if it is damaged or not working. Try to identify and solve the problem using the troubleshooting table. If the problem continues to exist or if the device is defective, contact the customer service department or your dealer as repairs can only be carried out with a special tool.

Never attempt to repair a defective electrical device yourself!

Problem	Possible cause	Solution
The lamp does not light up.	The device is not switched on.	Press the On/Off switch.
	The device is not supplied with power.	Is the mains adapter properly connected? Check that the mains adapter is properly connected to the plug socket and that the barrel connector is properly inserted in the device and adjust if required.
		Is the plug socket defective? Try another mains socket.
		Is there mains voltage? Check the fuse of the mains connection.
The LEDs have exceeded their lifespan.	Is the mains cable or the mains adapter defective? Contact the customer service department or your specialist dealer.	The device needs to be repaired. Contact the customer service department or your specialist dealer.
	The LEDs are defective.	The device needs to be repaired. Contact the customer service department or your specialist dealer.

Technical data

Article number	04379
Model number of device	LND018915V1
Dimensions (W x H x D)	260 x 150 x 20 mm
Weight	500 g
Light source	LEDs
Light intensity	> 10 000 Lux at a safety distance of 10 cm
Model number of mains adapter	CS12F120100FGF / CS12F120100FBF
Power supply	12 V DC, 1 A
Operating conditions	0 to +40 °C at a relative humidity of 30 – 90 % Atmospheric pressure: 700 – 1060 hPa
Storage and transportation conditions	-10 to +60 °C and a relative humidity of 30 – 90 % Atmospheric pressure: 700 – 1060 hPa
Product classification	1. Protection class II 2. No application parts 3. Protection against water ingress: IP21 4. No category AP / APG equipment 5. Operating mode: Continuous operation
Maximum radiation output	14 Wm ⁻² sr ⁻¹
Light emission	Illuminated area
Irradiated area	Approx. 400 cm ²
Risk group light source devices	Exception group classified in accordance with IEC 60601-2-57:2011
Product lifespan	5 years
ID instructions	Z 04379 M DS V1 1018

Subject to technical modifications.

Safety and performance standards

The LED light therapy lamp has been developed in compliance with the highest safety and performance standards including electromagnetic compatibility (EMC). The LED light therapy lamp meets the requirements of the following regulations:

The device meets the requirements of the Medical Devices Act (MPG) and the Medical Device Directive MDD 92/43/EEC

Notes on EMC


- The device is subject to specific precautionary measures with regard to its electromagnetic compatibility. When installing the device and starting to operate it, make sure to observe the EMC notes under “Normative references”.
- The device must be installed and put into operation for the first time in accordance with the provided documentation.
- Cordless communications systems such as devices for private WLAN networks, mobile phones, wireless telephones with base station and walkie-talkies may interfere with the device and should be kept at a minimum safety distance of (d). The safety distances (d) for frequencies from 800 MHz to 2.5 GHz as recommended by the manufacturer are listed in the table “Recommended safety distance between-portable and mobile RF communications systems and model LND018915V1” in the chapter “Normative references”.

Disposal

Please dispose of the packaging material in an environmentally friendly manner and take it to a recycling centre.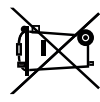

Please be aware that this device bears the WEEE symbol (see illustration). The symbol denotes that the device may not be disposed of together with normal household waste but must be properly disposed of in accordance with the regulations for electronic waste.

If you have any questions on this, please contact your local waste disposal authority.

Normative references

Guidelines and manufacturer's declaration – Electromagnetic immunity			
Model LND018915V1 is designed for use in an environment as set out below. The customer or user of Model LND018915V1 must ensure that the device is operated in such an environment.			
Immunity test	IEC 60601 Test level	Compliance level	Electromagnetic environment – Guidelines
Electrostatic discharge (ESD) IEC 61000-4-2	±6 kV contact discharge ±8 kV air discharge	±6 kV contact discharge ±8 kV air discharge	The surface on which the device stands should be made of wood, concrete or ceramic tiles. If the surface on which the device stands is covered with synthetic material, the minimum relative humidity must be 30 %.
Electrical fast transient interferences / bursts IEC 61000-4-4	±2 kV for power cables	±2 kV for power cables	The quality of the voltage supply must correspond to a standard commercial or hospital environment.
Surge voltage IEC 61000-4-5	±1 kV normal mode voltage	±1 kV normal mode voltage	The quality of the voltage supply must correspond to a standard commercial or hospital environment.

Voltage dips, short interruptions and supply voltage fluctuations IEC 61000-4-11	<p><5 % U_T (>95 % U_T dip) for 0.5 periods</p> <p>40 % U_T (60 % U_T dip) for 5 periods</p> <p>70 % U_T (30 % U_T dip) for 25 periods</p> <p><5 % U_T (>95 % U_T dip) for 5 seconds</p> <p>3 A/m</p>	<p><5 % U_T (>95 % U_T dip) for 0.5 periods</p> <p>40 % U_T (60 % U_T dip) for 5 periods</p> <p>70 % U_T (30 % U_T dip) for 25 periods</p> <p><5 % U_T (>95 % U_T dip) for 5 seconds</p> <p>3 A/m</p>	<p>The quality of the voltage supply must correspond to a standard commercial or hospital environment. If it is necessary to ensure that Model LND018915V1 continues to operate despite mains supply interruptions, we recommend connecting Model LND018915V1 to an uninterruptable power supply.</p>
Magnetic field at supply frequency (50/60 Hz) IEC 61000-4-8	Magnetic fields at the mains frequency should correspond to the typical levels for a commercial and hospital environment.		
NOTE: U_T stands for the alternating mains voltage prior to the application of the test levels.			

Guidelines and manufacturer's declaration – Electromagnetic immunity

Model LND018915V1 is designed for use in an environment as set out below. The customer or user of Model LND018915V1 must ensure that the device is operated in such an environment.

Immunity test	IEC 60601 Test level	Compliance level	Electromagnetic environment – Guidelines
Conducted RF interferences IEC 61000-4-6	3 V _{eff} 150 kHz to 80 MHz	3 V	<p>Portable and mobile radio devices should not be used closer to Model LND018915V1 (including the cables) than the recommended safety distance, which is calculated using the equation for transmission frequency.</p> <p>Recommended safety distance</p> <p>$d = 1.167 \sqrt{P}$ $d = 1.167 \sqrt{P}$ 80 MHz to 800 MHz $d = 2.333 \sqrt{P}$ 800 MHz to 2.5 GHz</p>
Radiated RF interferences IEC 61000-4-3	3 V/m 80 MHz to 2.5 GHz	3 V/m	<p>P stands for the maximum output of the transmitter in watts (W) in accordance with the specifications of the manufacturer of the transmitter and d is the recommended safety distance in metres (m).</p> <p>According to an on-site test ^a, the field strength of fixed transmitters should, for all frequencies, be lower than the compliance level.^b Interferences may occur in the vicinity of devices with the following symbol:</p> 

NOTE 1: The higher frequency range applies at 80 Hz and 800 MHz.

NOTE 2: These guidelines may not apply in all cases. The propagation of electromagnetic variables is influenced by absorption and reflection by buildings, objects and people.

^a It is not possible to theoretically predetermine the field strength of stationary transmitters, for instance for the base stations of wireless phones and mobile land mobile radios, amateur radio stations, AM and FM radio and TV broadcasting stations. In order to determine the electromagnetic environment with regard to stationary transmitters, a site evaluation should be considered. If the measured field strength exceeds the maximum compliance level at the site at which Model LND018915V1 is used, Model LND018915V1 should be monitored in order to demonstrate the provision of the intended functions. If abnormal performance characteristics are observed, additional measures may become necessary; e.g., the direction in which the device faces or the location of Model LND018915V1 could be changed.

^b Throughout the frequency range of 150 kHz to 80 MHz, the field strength should be under [V1] V/m

Guidelines and manufacturer's declaration – Electromagnetic emissions

Model LND018915V1 is designed for operation in an environment as specified below. The customer or user of Model LND018915V1 must ensure that the device is operated in such an environment.

Interference emission measurement	Compliance	Electromagnetic environment – guidelines
RF emission CISPR 11	Group 1	Model LND018915V1 uses RF energy solely for its internal functions. For this reason, the RF emissions are extremely low, meaning that interferences affecting neighbouring devices are unlikely.
RF emission CISPR 11	Class B	Model LND018915V1 is suitable for use in all facilities, including homes and establishments which are directly connected to the public low voltage supply network for private households.
Emission of harmonic currents IEC 61000-3-2	Class A	
Emission of voltage fluctuations / flickering IEC 61000-3-3	Met	

Recommended safety distance between portable and mobile RF communications systems and Model LND018915V1

Model LND018915V1 is intended for use in an electromagnetic environment in which radiated RF interferences are controlled. The customer or user of Model LND018915V1 can help to prevent electromagnetic interference by ensuring that the minimum distance between portable and mobile RF communications systems (transmitters) and Model LND018915V1 recommended below is complied with; this minimum distance is based on the maximum output of the communications system.

Maximum emitted output of the transmitter W	Safety distance according to transmission frequency m		
	150 kHz to 80 MHz $d = \left[\frac{3,5}{V_1}\right]\sqrt{P}$	80 MHz to 800 MHz $d = \left[\frac{3,5}{E_1}\right]\sqrt{P}$	800 MHz to 2.5 GHz $d = \left[\frac{7}{E_1}\right]\sqrt{P}$
0.01	0.117	0.117	0.233
0.1	0.369	0.369	0.738
1	1.167	1.167	2.333
10	3.690	3.690	7.377
100	11.67	11.67	23.33

For transmitters whose maximum output is not specified above, the recommended safety distance d in meters (m) can be estimated using the relevant equation for transmission frequency, where P is the maximum output of the transmitter in watts (W) as specified by the manufacturer of the transmitter.

NOTE 1: At 80 Hz and 800 MHz, the safety distance applies for the higher frequency range.

NOTE 2: These guidelines may not apply in all cases. The propagation of electromagnetic variables is influenced by absorption and reflection by buildings, objects and people.

The LED light therapy lamp is manufactured by:

Zero-Plus International Limited

Room 1004, 10/F., Join-In Hang Sing Centre, 71-75 Container Port Road, Kwai Chung, New Territories, Hong Kong

The LED light therapy lamp is marketed by:

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Am Heisterbusch 1, 19258 Gallin, Germany

Tel.: +49 (0) 38851 314337* (Mo. – Fr. 8 AM – 12 PM) • <http://www.dspro.de/kontakt>

* Calls to German landlines are subject to charges. The cost varies, depending on the service provider.

EC	REP
-----------	------------

EU representative in accordance with Directive 93/42 EEC:

I-Plus Company Ltd.

4B Christchurch House, Beaufort Court, Medway City Estate, Rochester, Kent. ME2 4FZ, UK

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DS Produkte GmbH
GlobalMIND GmbH