## Anleitung

Artikelnummer: Z 07402

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Product number: Z 07402

Dear customer,

We are delighted that you have decided to purchase our **ankle bandage**.

If you have any questions, please contact our customer service via our website: [www.service-shopping.de](http://www.service-shopping.de)

### Proper use

- The product is suitable for short-time stabilising the ankle joint.
- The product is not suitable for therapeutic purposes. Consult your GP or orthopaedist before using the product.
- The product can be worn both on the left and right foot and is suitable for all foot sizes.
- Suitable to be worn when doing sports, running or at home.
- Only use the product for the intended purpose. Any other use is deemed improper.
- Defects caused by improper handling, damage or failure to observe the care notes are excluded from the warranty. This also applies to normal wear and tear.

### Safety instructions

- Risk of suffocation! Keep the product and the packaging material away from children and animals.
- The product is only intended to support the ankle joint for short time. It is not suitable for therapeutic or medical applications. Consult your physician in case of persistent pain.
- The product is not suitable for treating bone fractures or torn ligaments etc. Consult your physician in case of acute injuries.
- Store the product in a dry, clean area that is out of the reach of children and animals and that is not exposed to direct sunlight.

### Material

55 % polyester, 30 % copper fibre, 15 % elastane

### Care

- |   |                         |   |               |
|---|-------------------------|---|---------------|
|  | Hand wash only          |  | Do not bleach |
|  | Do not dry in the dryer |  | Do not iron   |
|  | Do not dry clean        |   |               |

### Putting on the bandage

#### CAUTION!

- The bandage may only be used for a short period of time. It is not suitable for constant use.
- If you experience any pain while using the product, take it off and consult your doctor if necessary.

1. Pull the bandage over your foot. The heel is intended to sit in the recess of the bandage.

### Disposal

The packaging material is recyclable. Dispose of the packaging in an environmentally-friendly manner and make it available to the collection service for recyclable materials.

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**[www.service-shopping.de](http://www.service-shopping.de)**

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All rights reserved.
DS Produkte GmbH, Am Heisterbusch 1, 19258 Gallin, Duitsland